

## GS02 - PUFF SWITCH USER MANUAL



# WARNINGS

- While removing your Puff Switch from your GlassOuse or other devices, pay attention not to damage the wire joining points both on the 3.5mm jack connector and the Bite Switch module parts.
- To change the battery, remove the soft silicone cover of Puff Switch module and the battery inside, and then place the new one. Please pay attention to "positive" (+) remark on the battery during replacement.
- Keep Puff Switch away from magnets.

#### PUFF SWITCH

Offering short circuit low signals, even though Puff Switch is genuinely designed for GlassOuse V1.2, you can connect it to any kind of adapted device offering a mono 3.5mm jack connector plug in for external switch controls. Other than regular sip and puff systems, Puff Switch uses an electronic system, which allows you to do clicking actions via simple puffs.

Puff Switch works with 125mAh 3V CEBA CR1632 coin size battery. The battery life can go up to two months and it is easy to change. The built-in microphone has a blow trigger converting the frequencies derived from puffing action into signals automatically to do the clicks. Puff Switch's smart battery system turns the device off in one hour if it is not used. The switch also can be turned off manually via pressing the small button placed on the module.



#### 

7th Floor, Huarong Building, Qiaolian East, Bulong Road, Longgang District, 518129, Shenzhen, Guangdong, China P. 0086-755-8920-5835 F. 0086-755-8980-5835-812 info@glassouse.com | www.glassouse.com | www.ennopro.com



### SWITCH CONNECTION

- 1. Plug Puff Switch to Glassouse V1.2 or any 3.5mm jack adapted device via 3.5mm connector.
- 2. Adjust the position of Puff Switch wire until you feel comfortable enough to do puffing action.
- 3. Press the power button on your switch to activate Puff Switch. The LED light will flash. Press the power button again to turn off your Puff Switch. The LED light will flash for three seconds.

# LED SIGNALS

The LED light on Puff Switch tells you the sensitivity via blinking speeds. Press and hold power button for 3 seconds. 1. High Sensitivity - The LED light blinks nine times in fast speed.

- 2. Medium Sensitivity The LED light blinks six times in medium speed.
- 3. Low Sensitivity The LED light blinks three times in low speed.

## GS02 - PUFF SWITCH

Lenght: 170mm x Width: 24.5mm x Thickness: 11mm Weight: 11g Cable Length: 110mm Battery Type: CR1632 3V coin size battery

# DEVICE



#### 

7th Floor, Huarong Building, Qiaolian East, Bulong Road, Longgang District, 518129, Shenzhen, Guangdong, China P. 0086-755-8920-5835 F. 0086-755-8980-5835-812 info@glassouse.com | www.glassouse.com | www.ennopro.com