

GlassOuse

Assistive Device


www.glassouse.com

GS07 Pillow Switch

User Manual

Press Switch GS07

Dimensions	Diameter: 96.9mm x Thickness: 31.55mm
Weight	73±2g
Cable Length	185±5cm
Activation Surface	86.30mm
Activation Force	150g

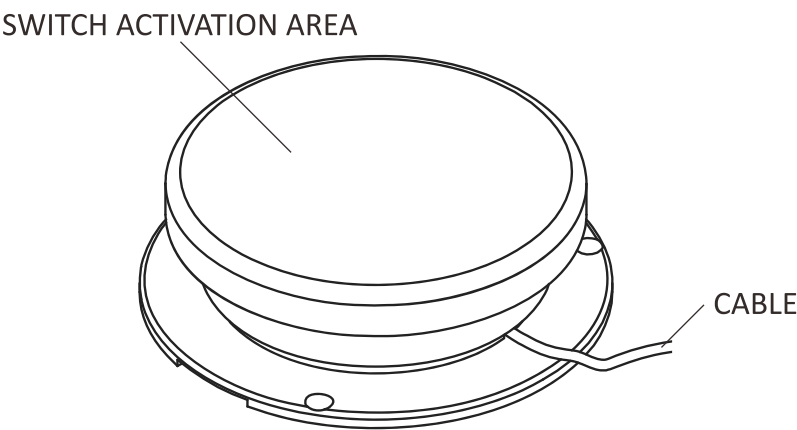

Pillow Switch

A durable switch option covered with soft cushion for users with less physical control. GS07 Pillow Switch provides 8.63cm activation surface, making the item comfortable enough for both user groups needing either a large or smaller target area to interact. Operated by 150g activation force, GS07 Pillow Switch has an auditory click and tactile feedback.

Pillow Switch comes with a 185cm long cable and a 3.5mm mono jack plug-in. Can be used with touch fastener straps and its attached safety pin to mount on fabric surfaces and wheelchair pads.

SWITCH CONNECTION

1. Plug GS07 Pillow Switch to GlassOuse or any 3.5mm jack adapted device via mono 3.5mm jack connector.

OR

- 2.1 Secure GS07 Pillow Switch on cushions or pillows via touch fastener straps.
- 2.2 Use the attached safety pin to mount on fabric surfaces and wheelchair pads.

WARNINGS

While removing your Pillow Switch from your GlassOuse or other devices, pay attention not to damage the wire joining points on the 3.5mm jack connector and do not pull the cable harshly.

Cushion cover can be removed and cleaned with laundry products.