

GlassOUSE

Assistive Device


www.glassouse.com

GS05S MINI PRESS SWITCH 120G

User Manual

GS05S Mini Press Switch 120g

Dimensions	Diameter: 49.2mm x Thickness: 25mm
Weight	24±2g
Cable Length	195±5cm
Activation Surface	40mm
Activation Force	120g

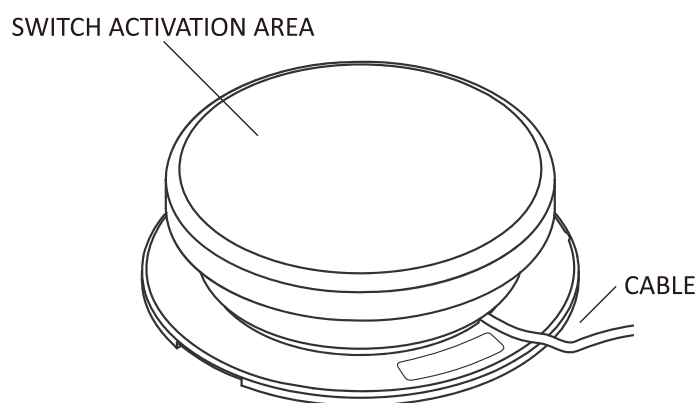

GS05S MINI PRESS SWITCH 120G

An ideal switch option for users with less physical control. Mini Press Switch provides 4cm activation surface, making the item comfortable enough for both user groups needing either a large or smaller target area to interact. Operated by 120g activation force, Mini Press Switch has an auditory click and tactile feedback.

Mini Press Switch comes with a 195cm long cable and a 3.5mm mono jack plug-in. Can be used with different mounting options or double-sided tapes.

SWITCH CONNECTION

1. Plug Mini Press Switch to GlassOuse or any 3.5mm jack adapted device via mono 3.5mm jack connector.
- 2.1 Place Mini Press Switch on a flat surface and adjust the position until you feel comfortable enough to do the pressing action before using double-sided tapes to secure the switch.

OR

- 2.2 Use a mounting option to secure the position of Mini Press Switch.

WARNINGS

While removing your Mini Press Switch from your GlassOuse or other devices, pay attention not to damage the wire joining points on the 3.5mm jack connector and do not pull the cable harshly.