

GlassOUSE

Assistive Device


www.glassouse.com

GS07S MINI PILLOW SWITCH

User Manual

GS07 Mini Pillow Switch

Dimensions	Diameter: 49.2mm x Thickness: 25mm
Weight	27.5±2g
Cable Length	195±5cm
Activation Surface	40mm
Activation Force	120g

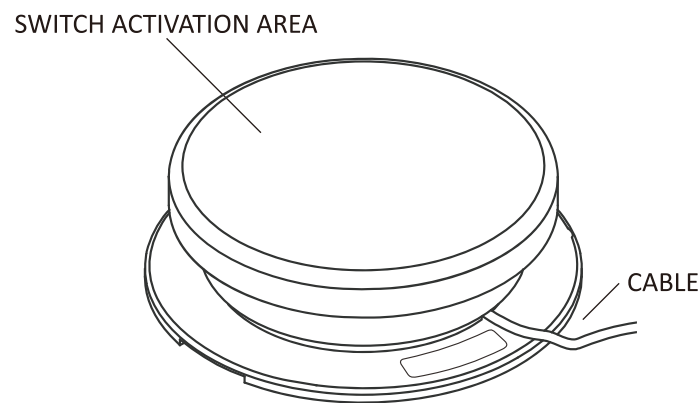

Mini Pillow Switch

A durable switch option covered with soft cushion for users with less physical control. Mini Pillow Switch provides 4cm activation surface, making the item comfortable enough for both user groups needing either a large or smaller target area to interact. Operated by 120g activation force, Pillow Switch has an auditory click and tactile feedback.

Mini Pillow Switch comes with a 195cm long cable and a 3.5mm mono jack plug-in. Can be used with touch fastener straps and its attached safety pin to mount on fabric surfaces and wheelchair pads.

SWITCH CONNECTION

1. Plug Mini Pillow Switch to GlassOuse or any 3.5mm jack adapted device via mono 3.5mm jack connector.

2.1 Secure Mini Pillow Switch on cushions or pillows via touch fastener straps.

OR

2.2 Use the attached safety pin to mount on fabric surfaces and wheelchair pads

WARNINGS

While removing your Mini Pillow Switch from your GlassOuse or other devices, pay attention not to damage the wire joining points on the 3.5mm jack connector and do not pull the cable harshly.

Cushion cover can be removed and cleaned with laundry products.