

GlassOUSE

Assistive Device


www.glassouse.com

GS08S MINI TOUCH SWITCH

User Manual

GS08S MINI TOUCH SWITCH

Dimensions	Diameter: 49.2mm x Thickness: 25mm
Weight	33 ±2g
Charging Port	USB-C
Adapter	5V1A

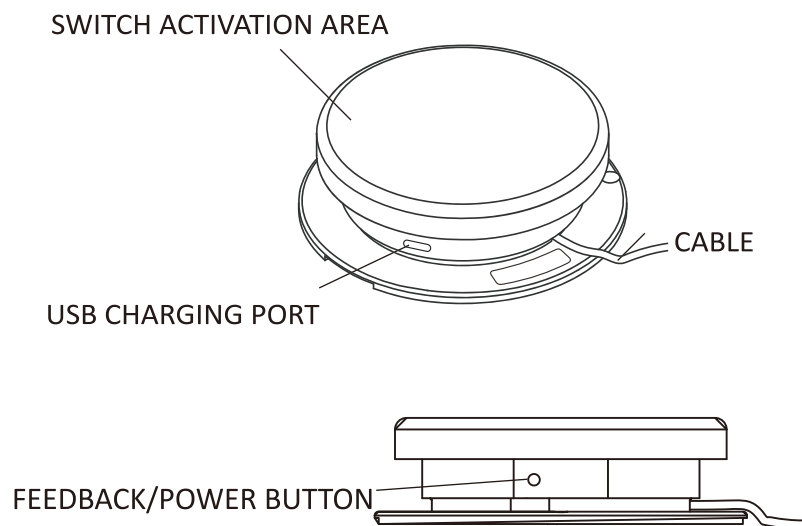

MINI TOUCH SWITCH

With an 4cm activation surface, Mini Touch Switch is highly sensitive to detect slight touches on activation area without any delay. Simply touch any area on the switch module to perform your clicks including the edges of the device. Mini Touch Switch comes with a 3.5mm mono jack connector plug and it is compatible with any kind of adapted devices besides GlassOuse .

CHARGING THE SWITCH

1. Insert a USB-C charging cable to charge the switch. The red LED indicator light will illuminate during charging.
2. Once fully charged, the red LED light will turn off, and the switch will automatically power off.
3. To turn ON the switch after charging, long press the POWER button for 2 seconds.

FEEDBACK SIGNALS

You can set the click feedback according to your needs. Use the tiny power button on the switch module for the desired feedback type as following. A second of press is enough to change the feedback mode.

1. Press the power button once – only visual LED feedback on without any auditory sound.
2. Press the power button once again – only auditory sound feedback on without no LED.
3. Press the power button once again – both visual LED and auditory sound feedback on together.
4. Press the power button once again – no visual LED or auditory sound feedback.

ACTIVATE/DEACTIVATE POWER SAVING MODE

To Deactivate Power Saving Mode: Press the power button five times. The device will confirm deactivation with five slow blinks of the green light. Note that the device will not shut down automatically and must be turned off manually.

To Activate Power Saving Mode: Press the power button five times. The device will confirm activation with five quick blinks of the green light. The device will power off automatically after 1 hour of inactivity. (Default: Power Saving Mode enabled)

SWITCH CONNECTION

1. Plug Mini Touch Switch to Glassouse any 3.5mm jack adapted device via 3.5mm connector.
2. Adjust the position of Touch Switch until you feel comfortable enough to touch the switch.
3. Press the power button on your switch for 3 seconds to activate Mini Touch Switch. The LED light will flash with beep sound. Press the power button again for 3 seconds to turn off your Mini Touch Switch. The LED light will flash with beep to indicate that the switch is turned off.

WARNINGS

1. While removing your Mini Touch Switch from your GlassOuse or other devices, pay attention not to damage the wire joining points both on the 3.5mm jack connector and the Mini Touch Switch module parts.
2. Keep the switch away from water.
3. Do not use the switch while it is charging.
4. Keep Mini Touch Switch away from magnets.

EXTERNAL SECONDARY SWITCH

You can combine your Mini Touch Switch with a tiny and more portable option to extend the using range. Simply plug the secondary switch and place it according to your needs.