

# GlassOuse V1.4

## USER MANUAL



### WARNING!

- Keep GlassOuse V1.4 away from magnets.
- GlassOuse V1.4 is foldable as regular glasses. Please fold it carefully.
- Use GlassOuse charging cable only with GlassOuse device.
- Keep away from wet surfaces and water.
- Charge and calibrate GlassOuse before first use.

## BLUETOOTH CONNECTION

GlassOuse V1.4 supports connection to Bluetooth 4.0 and below.

- Activate Bluetooth on your device (for iOS and iPadOS, also activate Accessibility > Touch > Assistive Touch).
- Press and hold the “Connect” button on the GlassOuse. The LED light will blink.
- Find and link to “**GlassOuse V1.4**” using your device’s Bluetooth scan screen.
- To use GlassOuse with other devices, press the “Connect” button again, or dismiss GlassOuse connection from your device or turn off previously connected device’s Bluetooth function.

## MOUSE USE

After you wear GlassOuse like regular glasses, the cursor will move according to your head movements. To move the cursor around the screen, move your head accordingly. Use your switch for clicking actions or the “dwell” feature of your device. When your head is in a centered and comfortable position towards the screen, the cursor should be in the middle.

**NOTE:** You can bring back the mouse cursor to the centre by pushing it to the edge of your device’s screen with your head and retrieving it back.

## CALIBRATION

We suggest doing the calibration before the first use or when connected with the new device. If the cursor moves abnormally (i.e. vibration, sliding etc.) calibration is recommended.

- Connect GlassOuse with your device via Bluetooth. Wait until the LED is continuously blue.
- Keep the GlassOuse in an inverted position on a flat solid surface to start automatic calibration.
- During calibration, you will notice LED blinking red and blue alternatively. Once the LED is continuously blue again, calibration is finished.

**NOTE:** Calibration should be performed within 3 minutes of successful connection with the device.

## MOUSE SENSITIVITY

You can adjust your GlassOuse V1.4 mouse sensitivity via two ways:

- To set your cursor speed on your device (PC, tablet, phone, TV):
  1. Go to “Cursor Speed” options of the device.
  2. Select the desired speed.
- To set the cursor speed via GlassOuse:
  1. Connect GlassOuse to a G-switch through mono 3.5mm jack plug. Press your G-switch and “Reset” buttons simultaneously (use the pin to press the reset button). If you are using a Puff Switch, press “Reset” button of your GlassOuse while you are puffing towards the switch.
  2. While pressing, release “Reset” button first and then release G-switch button. The LED should blink red. If you are using a Puff Switch, release “Reset” button first and stop puffing after.

## G-SWITCH OPTIONS

As option to GlassOuse V1.4 compatible parts, G-Switch Series adaptive switches come with a 3.5mm jack plug connector to combine perfectly with GlassOuse V1.4. Alternatively, you can use it with the “dwell” feature of your device instead. Offering short circuit low signals, all G-Switch options are specially designed for GlassOuse V1.4 but if you have any other switch devices using 3.5mm connector, you can connect those to your GlassOuse too.

## LED INDICATOR

- **Charging:** **Red** LED is seen continuously on the device
- **Charging is complete:** No LED light on the device
- **Bluetooth is searching for a connection:** LED *blinks* **Blue**
- **Bluetooth connection is active:** LED is continuously **Blue**
- **Low battery:** LED *blinks* **Red**

## OS COMPATIBILITY

- **Computer** (Windows, Linux, macOS, Chrome OS)
- **Tablet** (Android, iPadOS13, Windows, Chrome OS)
- **Mobile** (Android, iOS13, Windows, Chrome OS)
- **SmartTV** (Android)

## BATTERY


GlassOuse V1.4 comes with CEBA 3.7V 360mAh Li-Po battery. The battery life can go up to 150 hours of use on single charge. Please charge the battery once every three months at least when you do not use your GlassOuse.

## ANGLE OF USE

GlassOuse V1.4 can be used at a 180° horizontal or 160° vertical angle. Please do not use it while reclining fully or looking at the ground.

## GLASSOUSE V1.4

- Length x Width x Thickness : (160x145x50)mm
- Weight: 42.5 GMS

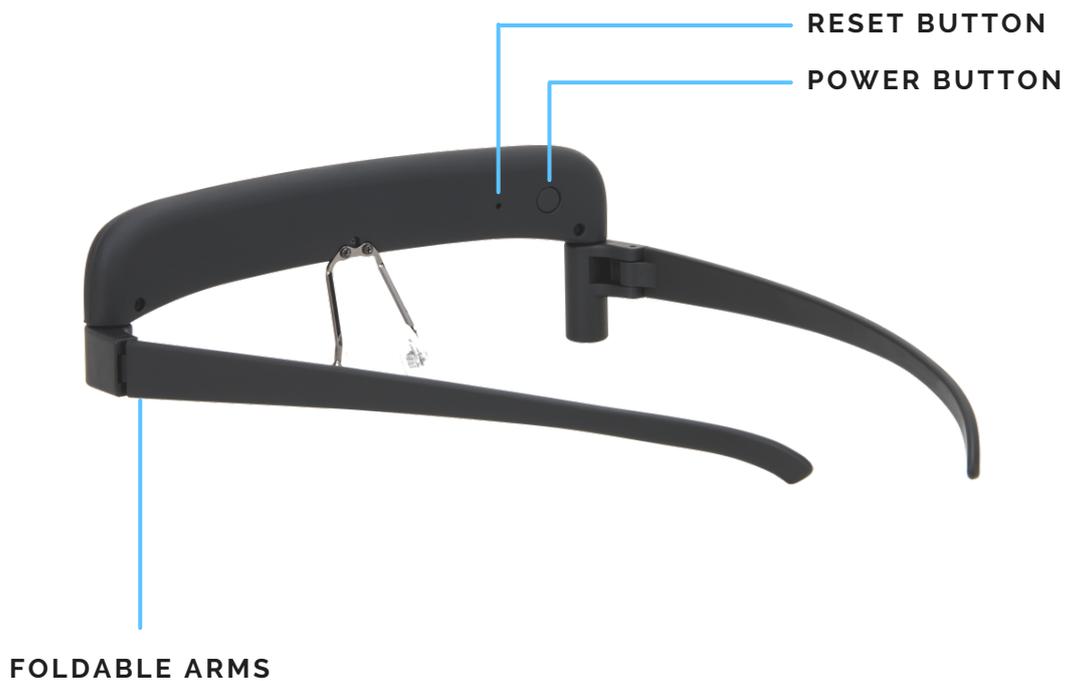

## WARRANTY

GlassOuse V1.4 comes with a 1-year international warranty against defects in materials and workmanship. For repair related queries, contact us at [support@glassouse.com](mailto:support@glassouse.com).