

GlassOuse

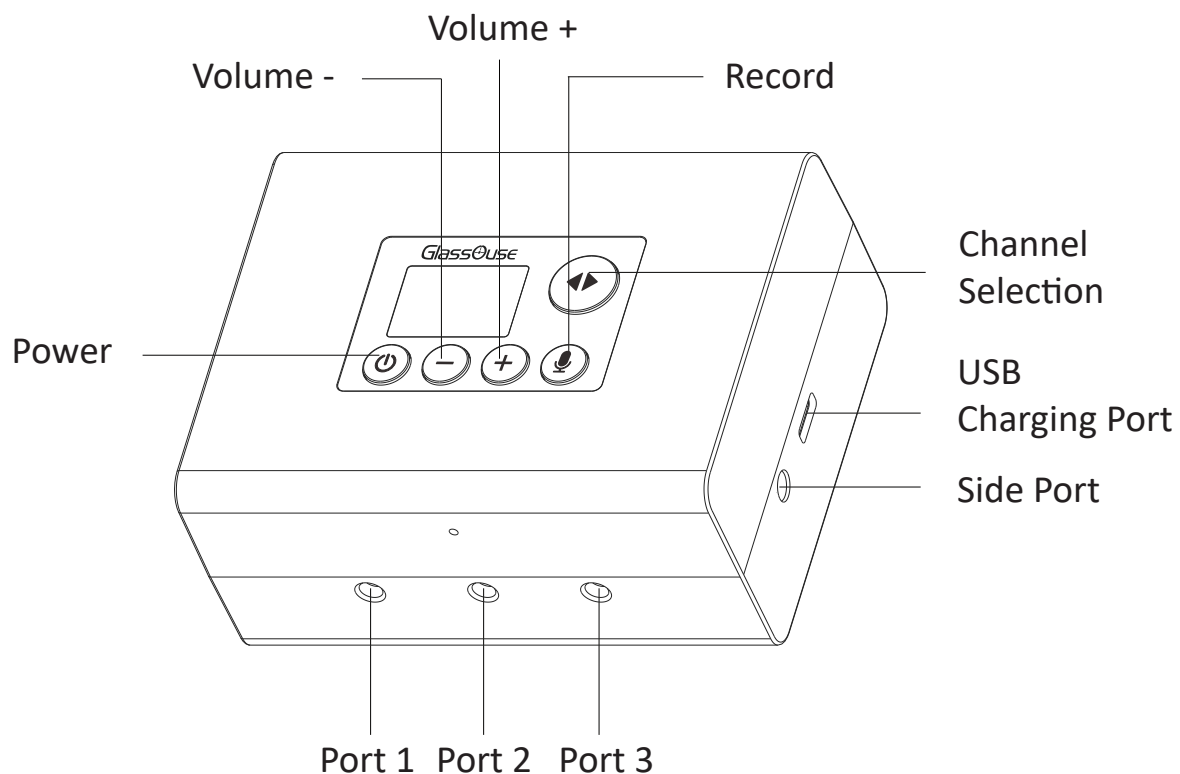
Assistive Device




www.glassouse.com

GT02-G-SPEECH

USER MANUAL

G-Speech



COMPONENTS	DESCRIPTION
Power Button“  ”	Turn ON/OFF the device
Volume “ - + ”	Increase/Decrease the volume
Record “  ”	Enter/exit recording mode
Channel Selection Button “  ”	Switch between five different channels
Front Port (1,2,3)	For Recoding & Playback (connect mono/stereo 3.5mm jack assistive switches)
Side Port (4)	For Channel Selection (connect mono/stereo 3.5mm jack assistive switches)
Type C Charging Port	Charge the device

1. INSIDE THE PACKAGE

Main unit *1 Charging cable *1 User manual *1

2. POWER ON/OFF



- To turn ON the device, long-press the Power Button for 3 seconds until the screen lights up and you hear a sound.
- To turn OFF the device, long-press the Power Button for 3 seconds until the screen turns off and you hear a sound.

3. ON SCREEN INDICATORS

- At the top of the screen, you will see five rectangles representing the available channels: 1, 2, 3, 4, and 5.
- On the second line of the screen, three elongated rectangles indicate the three headphone jack connections on the front panel.

NOTE: Each 3.5 mm headphone jack can be expanded to two connections using a Y-Splitter connector.



- If no recordings are stored on the device, the second line of the screen will display only the numbers 1, 2, and 3.
- When recordings are available:
 - If two commands are recorded using G-Switch on Port 1, the screen will show “1-A-B” in the first rectangle on the second line.
 - If two commands are recorded on Port 2, it will display “2-A-B”, and for Port 3, “3-A-B” will be shown.

- Each channel can store up to six recordings, with a total capacity of 30 recordings across all five channels.
- The third line of the screen displays three icons:
 - “▶” A triangle for playback status
 - “” A battery icon for battery level
 - “” Another triangle for volume level

4. CONNECT AND RECORD

- **Connect a Switch** – Insert a G-Switch into any of the three 3.5mm headphone jacks on the front panel.
 - **Before Recording** – The screen will display a triangle symbol.
 - **To Record** – Press the Record button on the device; the “▶ triangle symbol will change to a “◎ hollow symbol”. Then, long-press the connected switch for three seconds to start recording. The “◎ hollow symbol” will turn into a “● solid circle”, indicating that recording is in progress.
 - **Stop and Save Recording** – Press the connected switch again to stop recording. The screen will display the hollow symbol, and the recording will be saved automatically. If the recording reaches the 30-second limit, it will be saved automatically as well.
 - **Exit Recording Mode** – Press the record button to exit recording mode. The hollow symbol will change to a triangle symbol.
- Note:** Each recording can be up to 30 seconds long.

5. PLAYBACK RECORDINGS

- **Start Playback** – Press the connected switch to play a recording. The “  ” on the screen will change to a “  ”, indicating playback is in progress.
- **Pause Playback** – Press the switch again to pause.
- **Auto Stop** – Playback will stop automatically when the recording finishes.
- **Switch Channels** – Press the Channel Selection button on the device or use a GS Series switch connected to the side port. Each press will cycle through channels 1, 2, 3, 4, and 5 in sequence.

6. ADJUST VOLUME

- **Press the Volume – and Volume + buttons** to increase or decrease the volume by 10% per press. The screen will display the current volume level.
- **For faster adjustments, long-press the Volume – or Volume + buttons.**

7. CLEAR RECORDINGS

- **Clear Channel Recordings** – To delete all recordings on the current channel, press and hold the Channel Selection button until you hear a beep. While holding it, press and hold the Volume (-) button for two seconds to confirm deletion.

- **Clear All Recordings** – To erase all saved recordings, press and hold the Channel Selection button until you hear a beep. While holding it, press and hold the Volume (+) button for two seconds to confirm deletion.

8. LOW BATTERY AND CHARGING

- **Low Battery** – The battery icon on the screen will blink when the battery is low.
- **Charging** – Connect the device to a power source using the Type-C charging port. During charging, the battery icon will blink and show a running signal.
- **Charging Complete** – Once fully charged, the battery icon will display a full battery symbol.

9. PRECAUTIONS

- When disconnecting the GS Series Switch, handle the 3.5mm plug and wire connection points carefully to avoid damage.
- Keep the device away from water, moisture, and liquids to prevent malfunctions.
- Do not expose the device to fire or high temperatures.
- Do not use the device while it is charging.
- Keep the device away from magnets to prevent interference or damage to internal components.