

GlassOUSE


Assistive Device

www.glassouse.com

GS14 New Bite Switch

User Manual

GS14 NEW BITE SWITCH

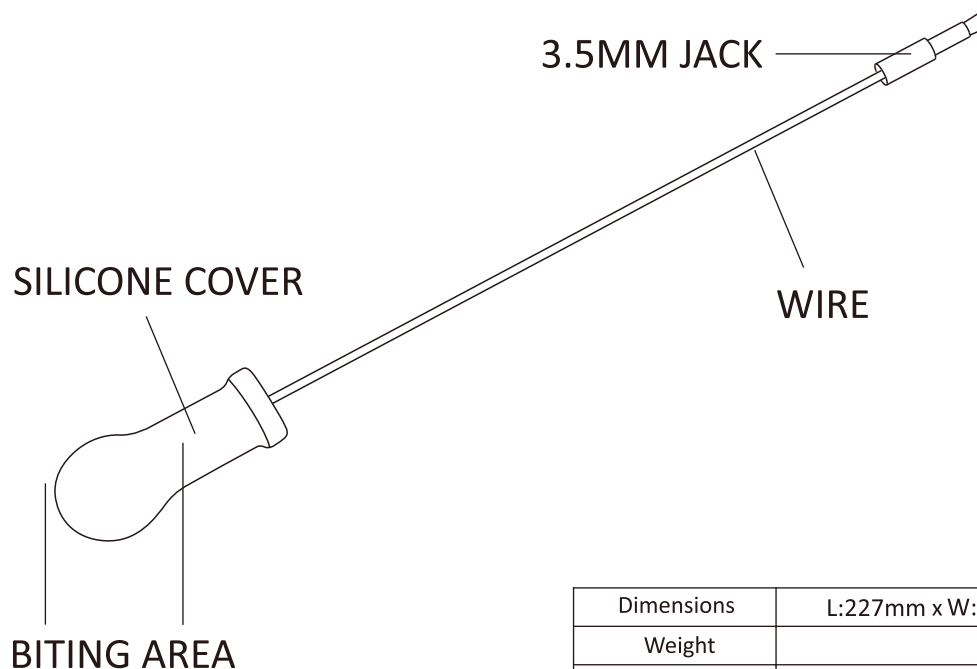

Dimensions	L:227mm x W:19.8mm x H:14mm
Weight	10.9g
Cable Length	154mm±5mm

INSIDE THE BOX

New Bite Switch

Replacement Silicone Tip

NEW BITE SWITCH

GlassOuse New Bite Switch is a first of its kind adaptive switch which is activated when you bite via your mouth or press it between your lips. Whenever this action is performed you can hear a click sound, sending the input signal to your device. It comes with a long 154 mm adjustable cable attached to its head and a 3.5mm jack connector on the other. The cable is flexible, allowing you to adjust the switch position as per your needs.

Though genuinely designed for GlassOuse, it can connect with any other adapted device offering a mono 3.5mm jack connector plug. New Bite Switch is made of strong and durable material while still being super lightweight - only 10.9g. The bite switch has been tested for 5,000,000 clicks under 3 tons of pressure (lab testing environment), strong enough to withhold strong teeth and jaw pressure.

SWITCH CONNECTION

1. Plug New Bite Switch to GlassOuse via 3.5mm jack.
2. Adjust the position of New Bite Switch wire until you feel comfortable enough to do biting action.
3. Press the switch to activate your device.

WARNINGS

1. While detaching your New Bite Switch from your GlassOuse or other devices, pay attention not to damage the wire joining points both on the 3.5mm jack connector and the New Bite Switch module parts.
2. Do not immerse New Bite Switch in water for cleaning purpose. Only clean silicone cover piece after removing it from the Bite Switch module.

Manufacturer: EnnoPro Group Limited

Address: 7th Floor, Huarong Building, Qiaolian East, Bulong Road, Longgang District, 518129, Shenzhen, Guangdong, China

Phone: 0086-755-89205835

Mail: support@glassouse.com

Web: www.glassouse.com